

# TREATMENT GUIDE



CLINICAL ASSESSMENT (if boxes are checked below, patient is at greater-risk)	
<input type="checkbox"/>	Inflammation/Bleeding
<input type="checkbox"/>	4mm probe depths or greater
<input type="checkbox"/>	Bone loss/CAL
<input type="checkbox"/>	Restorative [Failing restorations/Abcesses]
<input type="checkbox"/>	Desaturation [Airway/Nitric oxide loss/Sleep disorder]

MEDICAL ASSESSMENT (if boxes are checked below, consider contraindications)	
<input type="checkbox"/>	Medical Risk [Pregnancy/Cardiovascular/Autoimmune/Cancer/etc]
<input type="checkbox"/>	Genetic Risk [9p21/MTHFR/HAP/4q25/KIF6/APOE]
<input type="checkbox"/>	Inflammatory Risk [MPO/PLA2/ADMA/CRP/Glucose-A1C]
<input type="checkbox"/>	Antibiotic Resistant Risk [Past history/1x or more a year]
<input type="checkbox"/>	Other:

## HR5™

## FAM™

KEY	<input type="checkbox"/> More Effective	<input type="checkbox"/> Moderately Effective	<input type="checkbox"/> Less Effective	Aa	Pg	Td	Tf	Fn	Pi	Cr	Cs	Ec
<b>TEST RESULTS</b>												
<b>MECHANICAL TREATMENTS</b>												
Scaling & Root Planing	<i>*resistant bacteria: Aa, Pg, Tf, Pi</i>											
Air-Flow [EMS/Hu-Friedy]												
Lasers / LANAP / Photomodulation												
Piezo / Ultrasonic [3-5 Tip]												
Periodontal Surgery [Removal of Infected Tissue]												
<b>ANTIBIOTIC TREATMENTS</b>												
Systemic Antibiotic [Prescription]	See Antibiotic Guide											
LAA: Minocycline [Arestin]												
LAA: Doxycycline [Vibramycin/Atridox]												
<b>ANTIMICROBIAL TREATMENTS</b>												
Molecular Iodine [IO rinse/gel/spray]												
Povidone Iodine [Betadine, Therasol]	<i>*resistant combinations Pg-Fn / Tf-Fn</i>											
Chlorites/Chlorides [Carifree, Oracare, CloSYS]	<i>*resistant combinations Pg-Fn / Tf-Fn</i>											
Chlorhexidine [PerioGuard, PerioMed]	<i>*resistant combinations Pg-Fn / Tf-Fn</i>											
OTC [Listerine, Crest, Scope, Colgate, etc]	<i>*resistant combinations Pg-Fn / Tf-Fn</i>											
Tray System [PerioProtect/PerioRestore] [1.7% H <sub>2</sub> O <sub>2</sub> ]	<i>*resistant combinations Pg-Fn / Tf-Fn</i>											
Ozone	<i>*resistant combinations Pg-Fn / Tf-Fn</i>											
<b>NATURAL PRODUCTS</b>												
StellaLife [Gel/Rinse/Spray]												
Essential oils [Biocidin Botanicals, Tooth & Gum]												
PerioSciences												
Zinc [Lozenges, rinses, spray]												
Xylitol [Xlear-Spry, Epic, Bite, Xylimelts, etc]												
<b>HOME CARE</b>												
Electric Toothbrush [Socicare/Oral B]												
Oral Irrigator [Waterpik/WaterFlosser]												
Interdental [GUM-softpicks/TePe/Floss/Airflosser]												
<b>PROBIOTICS &amp; GI HEALTH</b>												
ProBioraPro [Oral Probiotic]												
Burst/ProDental/Pharmaden/ Riven [Oral Probiotic]												
Ortho Molecular, BioGaia, Hyperbiotics, Seed, Biocidin [GI]												
Peptide BPC 157, Glutamine, Sodium butyrate [GI]												